# Antimicrobial Dermal Template

## EN Product description

**Endoform® Antimicrobial Dermal Template** is an advanced wound care dressing primarily composed of natural, non-reconstituted collagen retaining the native extracellular matrix associated macromolecules including fibronectin, glycosaminoglycans, laminin, and elastin. The Dermal Template is supplied as sterile intact or fenestrated sheets.

**Endoform® Antimicrobial Dermal Template** contains approximately 12 µg/cm<sup>2</sup> (~0.3% w/w) ionic silver, intended to inhibit colonization of the dressing. The dressing is effective against a broad spectrum of microbes, including *Acinetobacter baumannii*, *Candida albicans*, *Candida parapsilosis*, *Candida glabrata*, *Escherichia coli*, Methicillin Resistant *Staphylococcus aureus* (MRSA), coagulase-negative *Staphylococci*, group A (beta-hemolytic) *Streptococci*, *Pseudomonas aeruginosa*, *Aspergillus niger*, and *Vancomycin Resistant Enterococci* (VRE). The dressing provides sustained antimicrobial effectiveness within the dressing for up to 7 days.

**Endoform® Antimicrobial Dermal Template** is derived from ovine (sheep) extracellular matrix and it retains the innate biological structure of the native extracellular matrix. When rehydrated with exudate or sterile saline, the Dermal Template transforms into a soft conforming sheet, which is naturally incorporated into the wound over time.

Not made with natural rubber latex.

## Intended use

**Endoform® Antimicrobial Dermal Template** is a sterile, single use ovine forestomach-derived extracellular matrix intended to cover, protect, and provide a moist wound environment.

## Indications for use

**Endoform® Antimicrobial Dermal Template** is indicated for the management of wounds including:

- Partial and full thickness wounds
- Pressure ulcers
- Venous ulcers
- Diabetic ulcers
- Chronic vascular ulcers
- Tunneled/ undermined wounds
- Surgical wounds (donor sites, grafts, post Moh's surgery, post laser surgery, podiatric, wound dehiscence)
- Traumatic wounds (abrasions, lacerations, first and second-degree burns, and skin tears)
- Draining wounds

**Rx** Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

## Contraindications

- **Endoform® Antimicrobial Dermal Template** is derived from an ovine (sheep) source and should not be used on patients with known sensitivity to ovine (sheep) derived material or ionic silver.
- **Endoform® Antimicrobial Dermal Template** is not indicated for use on third degree burns.

## Precautions

- **Endoform® Antimicrobial Dermal Template** may be used when visible signs of infection are present in the wound area, only

when proper medical treatment addresses the underlying cause. Endoform Antimicrobial® Dermal Template is not intended to be a substitute for appropriate treatment of infection.

- **Endoform® Antimicrobial Dermal Template** is supplied as a sterile dressing. Do not use if the pouch seal is broken.
- Discard the device if mishandling has caused possible damage or contamination.
- Single use product. Do not attempt to re-sterilize. Discard all unused portions. Do not reuse.
- **Endoform® Antimicrobial Dermal Template**, like other silver-containing products, may cause transient discoloration of the wound bed and surrounding skin. Frequent or prolonged use of this product may result in permanent discoloration of the skin.
- Reduction in colonization or microbial growth on the device has not been shown to correlate with a reduction in infections in patients. Clinical studies to evaluate reduction in infection have not been performed.

## Storage

**Endoform® Antimicrobial Dermal Template** should be stored between 15°C/59°F–40°C/104°F in a clean and dry area, away from direct light.

## Appearance

**Endoform® Antimicrobial Dermal Template**, like other silver-containing products, may darken upon storage, after hydration in saline, when exposed to light, or when in contact with body fluids and tissues. This darkening does not affect product performance.

## Directions for use

These recommendations are designed to serve only as a general guideline. They are not intended to supersede institutional protocols or professional clinical judgment concerning patient care.

### 1) Wound Preparation

- Prepare the wound bed by cleansing, irrigation and if necessary debridement to ensure the wound is free of debris, necrotic tissue or infected tissue.

### 2) Dressing Application

- Select a sheet of the dermal template which is slightly larger than the wound and apply aseptically.
- Endoform® Antimicrobial Dermal Template** can be applied as a whole sheet or trimmed so that it contacts the wound margins. Multiple sheets can be used to cover the entire wound bed.
- For ease of handling, apply the dermal template by placing the dry material in the wound and rehydrating with exudate or sterile saline. When hydrated **Endoform® Antimicrobial Dermal Template** transforms into a soft conforming sheet. Ensure that the dermal template conforms to the underlying wound bed.
- To protect **Endoform® Antimicrobial Dermal Template** from adhering to the secondary dressing, consider applying a non-adherent dressing over the dermal template to help protect the tissue while facilitating an optimal moist wound healing environment.
- Secure the dressing using an appropriate secondary dressing. The overlying dressing should be changed according to standard of care and taking into account the level of exudate.
- The biodegradable **Endoform® Antimicrobial Dermal Template** is naturally absorbed and incorporated into the wound over time. **Endoform® Antimicrobial Dermal Template** may last up to 7 days in the wound, depending on the wound environment.

### 3) Dressing Changes

- Duration of treatment and reapplication is determined by the physician and depends upon the wound type and conditions.
- Reapply when **Endoform® Antimicrobial Dermal Template** has been incorporated into the wound. It is recommended to assess the wound within 72 hours. **Endoform® Antimicrobial Dermal Template** may last up to 7 days in the wound, depending on the wound environment
- Carefully cleanse the wound surface in accordance with established procedures. Do not attempt to remove the off-white to golden gel that forms from residual **Endoform® Antimicrobial Dermal Template** in the wound as it contains extracellular matrix components.
- If areas of the dermal template are dry upon inspection, rehydrate with sterile saline and leave in place
- It is not necessary to remove any residual **Endoform® Antimicrobial Dermal Template** during dressing changes. However, if the product has been overlapped onto the peri-wound area, the remaining loose product that has not incorporated into the wound may be gently removed around the edges if desired.
- Change the secondary dressings as needed, and when **Endoform® Antimicrobial Dermal Template** is re-applied.

## Symbols

ISO 15223-1: 2016 Medical devices – Symbols to be used with medical device labels, labeling and information to be supplied – Part 1: General requirements		
	5.4.2	Do not reuse
	5.2.6	Do not re-sterilize
	5.2.8	Do not use if package is damaged
	5.1.4	Use by expiration date YYYY/MM/DD
	5.3.4	Keep dry
	5.2.3	Sterilized with ethylene oxide
	5.3.7	Store at room temperature
	5.4.3	Consult instructions for use
	5.3.2	Keep away from heat and light
	-	Number of Units

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