

Use **Endoform**[®] and **Hydrofera Blue**[™] together to overcome stalled wound healing

APPLICATION GUIDE

Indicates and manages wound protease levels



- Non-cytotoxic
- Biofilm prevention
- ECM scaffold to build tissue
- Broad spectrum and long-acting antimicrobial activity

OPTIMIZED HEALING

Indicates and manages bioburden



- Non-cytotoxic
- Natural negative pressure, wicks bacteria laden exudate
- Natural autolytic debridement
- Flattens rolled wound edges
- Broad spectrum antibacterial protection

”

The combination of Hydrofera Blue and Endoform gives wounds that 1-2 punch that's needed to jumpstart the healing process.

As Endoform Antimicrobial and Natural work to indicate and control the proteases, and provide a scaffold to build tissue, Hydrofera Blue wicks away bacteria laden slough, debris and exudate while pulling healing agents to the wound bed.

Both products work to prevent the formation of biofilm, creating the optimal environment to prevent stalling and speed wound healing.

DR. JAMES STILLERMAN MD, CWSP


Use Endoform® and Hydrofera Blue™ together to overcome stalled wound healing

APPLICATION GUIDE

1


WOUND BED PREP

Sharp or Autolytic debridement (Hydrofera Blue CLASSIC aids in autolytic debridement).

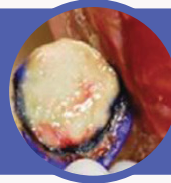

2

On a clean wound bed place Endoform® Antimicrobial covered with Hydrofera Blue CLASSIC twice per week until exudate levels have decreased.

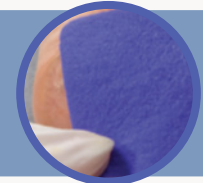

3

At the next dressing change use Endoform® Antimicrobial covered with Hydrofera Blue READY/READY-Transfer. Leave in place for up to 7 days



4

Repeat above application until significant improvement is observed.

5

As wound is healing switch to Endoform® Natural covered with Hydrofera Blue READY/READY-Transfer.




6

Repeat above step until wound is closed.

!

IF RECURRENCE OF INFECTION OCCURS (Hydrofera Blue fades to white), apply Endoform® Antimicrobial.

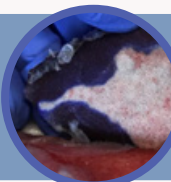

Choose the **Appulse system of healing** to help drive early wound closure

RX Only. Prior to use, be sure to read the entire **Instructions For Use** package insert supplied with the products. For product questions, sampling needs, or detailed clinical questions concerning our products in the US, please call **1-860-337-7730** or visit our website: **www.appulsemed.com**.

Dr Stillerman has a consultancy agreement with Aroa Biosurgery and Hydrofera LLC.

Endoform® is a registered trademark of Aroa Biosurgery Limited. Hydrofera Blue and the Hydrofera Blue logo are trademarks of Hydrofera, LLC. ©2019. MKT. 1518.00