



Partial and Deep Partial Burns

Clinical Evidence



Every body has the power to heal

www.endoform.com

Partial and Deep Partial Burns

Clinical Evidence

Partial Scald Burn

26-Year-old male, sustained partial scald to right hand. Surgical debridement performed 10 days prior to Endoform[®]. Evidence of incorporation of Endoform[®] at Day 10, and 100% epithelialization at Day 17.



Day 1 - pre-debridement



Day 10



Day 17

Aballay, A. and E. Tolchin (2018). Results of an ovine extracellular matrix dressing in the management of a variety of burns and wounds. The Symposium on Advanced Wound Care Charlotte, NC.



Bilateral Deep Partial Burn

77-Year-old male. Diabetic with peripheral neuropathy. Sustained bilateral deep partial burns from prolonged standing on hot surface. Hospitalized, debridement's and three previous treatments with CTP's. Endoform[®] initiated to close remaining areas.



Week 0



Week 1



Week 3



Week 4



Week 11

Zilberman, I. (2015). Case Study 19: Bilateral Feet plus Second Degree Burn, Hollister Incorporated.

Partial and Deep Partial Burns Clinical Evidence

Electrical Burn

21-Year-old male, sustained deep partial electrical burn to left foot. Surgical debridement prior to Endoform[®]. 100% Epithelialization at week 4.



Week 0



Week 3



Week 4

Aballay, A. and E. Tolchin (2018). Results of an ovine extracellular matrix dressing in the management of a variety of burns and wounds. The Symposium on Advanced Wound Care Charlotte, NC.



Deep Partial Thermal Burn

60-Year-old male. Sustained deep partial burn during house fire. Initiated Endoform Antimicrobial[®] treatment 1 week post injury. 100% Epithelialization by Day 9.



Day 1



Day 7



Day 8



Day 9



7 Months

Case courtesy of Achabal P. RN, MSC, CWON (2019).

Partial and Deep Partial Burns Clinical Evidence

Pediatric Burn

9-Month-old male. Flame burn to face with significant pain and discomfort.



Week 0



Week 2



Week 3



Week 3 - post debridement



Week 7



Week 8

Case courtesy of Rami Al Mousa, MD, Al Isra Hospital Amman, Jordan (2019).

RX Only. Prior to use, be sure to read the entire Instructions For Use package insert supplied with the product.

For product questions, sampling needs, or detailed clinical questions concerning our products in the US, please call 1-877-627-6224.

Endoform[®] is a registered trademark of Aroa Biosurgery Limited.



Endoform[®] Natural and Endoform[®] Antimicrobial Dermal Template are marketed in North America by Appulse.
www.appulsemed.com



AROA

Manufactured for:

AROA BIOSURGERY INC

7220 Trade Street, Suite 306, San Diego, CA
92121

1-877-627-6224

www.endoform.com

MKT.1442.02 | January 2020