



Doctor's Name: _____

Patient's Name: _____

Today's Date: _____

Date Of Next Appointment: _____

User information sheet

Your provider has identified that your wound needs a more advanced product.

Endoform® is a unique extracellular matrix (ECM) with a mix of molecules known to be important in healing.

The components of **Endoform**® help to control factors that slow down healing and later on can help to build new tissue.

Use as recommended by your healthcare professional (HCP).



For more information on **Endoform**®, scan the QR code to visit our website: www.endoform.com



Endoform® Application Information:

Typically **Endoform**® needs to be applied more frequently at the beginning of treatment and a course will extend over several weeks. **Endoform**® may need to be applied between visits as directed by your Healthcare Provider. **Endoform**® may also provide helpful information on the progress of your treatment. By examining the presence or absence of residual **Endoform**® in your wound bed your Healthcare Provider can assess how much **Endoform**® should be applied at each reapplication.



Alert

If your wound has an **unusually high level of wound drainage or pain** and is **hot or swollen**, please contact your health care provider.

Endoform® Product Selection:

Endoform® is available in two versions and your provider will select one or both during the course depending on how healing progresses.

As you receive supplies through home health providers or wound product suppliers, it is important to ensure you continue using **Endoform**® until your healthcare professional recommends a change.

Product versions and sizes may vary during the course of treatment, as determined by your HCP.

Endoform® Natural



Endoform® Antimicrobial




Apply (number) _____ piece(s) of **Endoform**® every _____ days

User information sheet

How to apply Endoform[®]:

1. Prepare the wound area for **Endoform**[®] application by gently rinsing.
2. Select a sheet of **Endoform**[®] which is larger than the wound. **Endoform**[®] can be applied as a whole sheet or folded so that it at least covers the wound edges. Apply to the wound bed without touching the area which will cover the wound. Ensure that **Endoform**[®] contacts the wound bed.
3. Hydrate **Endoform**[®] with sterile saline for 60 seconds.
4. Ensure that a secondary dressing is applied on top of **Endoform**[®].
5. Secure dressings in place with a suitable bandage or tape.

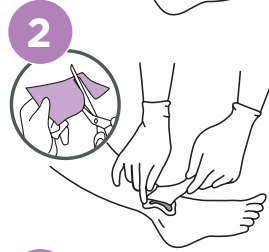

These instructions do not replace the specific instructions from your healthcare professional for the care of your wound. Please use as instructed and directed by your healthcare professional.



For more information on applying **Endoform**[®] scan the QR code.
www.qrs.ly/cm9z7vu



How to improve your healing success

If your wound has an **unusually high level of wound drainage or pain** and is **hot or swollen**, please contact your health care provider.



Keeping the wound clean



Managing other conditions:



Smoking cessation?



Other notes:

Patient Video Resources

www.qrs.ly/fo9yrb6



Endoform[®] Product Information

For product information visit our website:
www.endoform.com



Important information

Contraindications: **Endoform**[®] Dermal Template dressings are derived from sheep tissue, don't use these products if you know you are sensitive to this. Don't use **Endoform**[®] Dermal Template dressings on third degree burns. **Precautions:** If your wound is infected, inflamed, has excessive exudate, or is bleeding, stop use and contact your healthcare provider. **Endoform**[®] Dermal Template dressings are provided sterile and are to be used only once. If the pouch seal is broken or if the dressing looks damaged, throw it away and select a new pouch. Do not try to re-sterilize or reuse the dressing. Always use clean hands when applying **Endoform**[®] Dermal Template dressings. **Endoform**[®] **Antimicrobial** Dermal Template dressings contain silver which may cause the surrounding area of the skin to temporarily change color. Long term use can cause this color change to be permanent. Don't use **Endoform**[®] **Antimicrobial** dressings on babies or small children unless prescribed by your healthcare provider.

For product questions please call toll-free 1-877-627-6224.



Endoform[®] Dermal Template is marketed in North America by Appulse. www.appulsemed.com



Manufactured for: **ARO BIOSURGERY INC**
 7220 Trade Street, Suite 306, San Diego, CA 92121
 1-877-627-6224 | www.endoform.com

MKT. 1517.00 | January 2020